



## **Phase 3 has indicated Training equipment can be shared ensuring equipment is cleaned REGULARLY throughout training.**

### **CPR MANIKINS**

At time of CPR component the below must be met (Manikins not to be used again at any other assessment in training):

1. First participant to complete the CPR components (with supplied clean manikin) required of them – participant to use gloves supplied
2. Once Assessment is complete, participant to remove manikin face, lung bag and gloves, and place in bag provided by trainer
3. Participant must then disinfect the manikin chest, dispose of the disinfectant wipe and move to designated area
4. Second participant will be given a bag containing a clean manikin face and lung bag and approach manikin
5. Participant is to put on gloves and replace/install the lung bag and face as directed by the trainer.
6. For the 2 person CPR - Participant number 1 does normal one operator CPR, Participant 2 comes in to do compressions, while person 1 moves to the head of the manikin to maintain distance. Participant number 1 to put on defib, then swap.